

# INSTRUCTIONS FOR USE

## **【Product Name】**

Filtering Half Mask

## **【Model No.】**

CC-02

## **【Standard】**

EN 149:2001+A1:2009 FFP2 NR to regulation EU 2016/425

## **【Primary Structure】**

Filtering Half Mask consists of ES cotton, non-woven cloth, melt blown cloth, nose clip, ear strap and retaining clip.

## **【Use Range】**


Filtering Half Mask is applied for covering the mouth, nose and jaw of users in daily life or in general environment to protect users from virus, PM2.5, formaldehyde, smog and industrial dust.

## **【Warning】**

- Before use the mask first verify that it is suitable for the intended use. The wearer must be adequately trained prior to use and ensuring the masks are proper fit.
- The mask is disposable and cannot be reused.
- Using the masks for someone with facial hair will cause leakage problem, if the facial hair not covered, unlikely to achieve seal.
- Discard and replace the mask if it becomes damaged or higher breathing resistance.
- Don't use the mask in explosive gas, heavily polluted environment, oxygen-deficient environment, fire environment and the underwater work.
- Patients with heart disease or other disease should discard it after wearing uncomfortably.
- Masks marked "NR" shall not be used for more than one shift.
- This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.
- Do not use in explosive atmospheres.
- It is strictly prohibited to use after the package is damaged.
- Masks cannot be washed with water, which will damage the filter material structure and cause penetration and destroy electrostatic filter material filtration. When storing the mask, it is also necessary to avoid deformation of the mask.
- Masks should not be sterilized in the microwave.

## **【Using Instructions】**

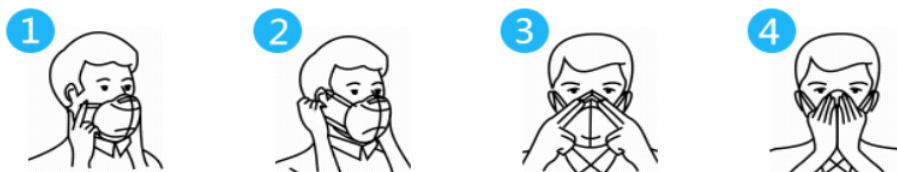


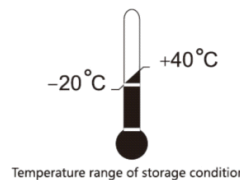


1. Open the particle filtering half mask, face to the inside of the mask, and hold the mask on each hand so that the nose clip is  
     at the top.
2. Hold the particle filtering half mask in position over the nose and mouth.
3. Adjust to a comfortable position and make the mask fit the face.
4. Bend the nose clip to make a tight seal around the nose.
5. Fit check
  - a) To test the fit of the particle filtering half mask, cup both hands over the particle filtering half mask and inhale sharply. If air flow is felt in the nose area, re-adjust/tighten the nose clip.
  - b) If flows are felt around the edges of the particle filtering half mask, re-position the mask harness to achieve a better fit.
6. Change the particle filtering half mask immediately if breathing becomes difficult or particle filtering half mask becomes  
     damaged or distorted.
7. Change the particle filtering half mask if a proper face seal cannot be achieved.
8. Careful observances of these instructions is an important step in safe particle filtering half mask use.



**【Storage Condition and Methods】**

1. The unused Masks needs to be stored in the environment that is well ventilated, dry and away from light. Keep away from fire, pollutants and potential source of pollution.
2. Storage temperature: -20 °C to 40°C and less than 80% relative humidity.
3. Keep the masks in original packaging boxes when during transportation to protected against mechanical damage and contamination.

**【Explanation about Signs, Symbols】**

	Single use		Keep out of direct sunlight
	Maximum relative humidity of storage conditions		The temperature range of storage conditions
	See the information provided by the manufacturer		CE mark and Notified Body No

**【The Manufactured date】**

See outer packing

**【The Expiry time】**

2 years

**【Manufacturer & Address】**

Manufacturer: Ningbo Haishu Changcheng Hygienic Material Factory

Add: Shanglin Village, Dongqiao Town, Haishu District, Ningbo City, Zhejiang Province, China

**【Telephone】**

Tel: +86-13646687448

**【Email】**

[529238270@qq.com](mailto:529238270@qq.com)

**【Website】**

<https://88392221.1688.com>

The EU DoC is to be delivered with accompanies the user information and masks together.

**【The Notified Body】**

**Universal Certification and Surveillance Service Trade Ltd. Co.**

Necip Fazıl Bulvarı Keyap Sitesi E2 Blok No: 44/84 Yukarı Dudullu Ümraniye-Istanbul

**NB:2163**